



### Step 1

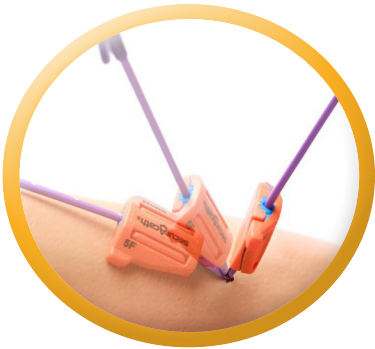

- Lift the line and SecurAcath® for 360 degree cleaning to be achieved on every dressing change.
- Do not twist or rotate the SecurAcath®.
- Lifting the SecurAcath® regularly may aid easy removal.

### Step 2

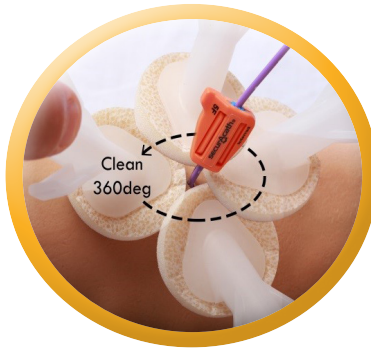

- Clean 360 degrees around the line and SecurAcath® with wipes or applicators as per Trust policy.

### Step 3



- Pay particular attention to the insertion site.
- The line will remain stable during cleaning.

### Step 4





- Clean the top and back of the SecurAcath®, the surrounding skin and line.
- Allow to air dry completely.
- Loosely apply a clear film dressing.

### Hints & Tips



- Use sterile forceps to remove debris between the line and the SecurAcath®.
- Dried blood can be lifted by applying saline soaked gauze for a few minutes
- If patient complains of acute pain from the SecurAcath®, the device may not be sited correctly and will need reviewing.
- SecurAcath® is bio-patch compatible.

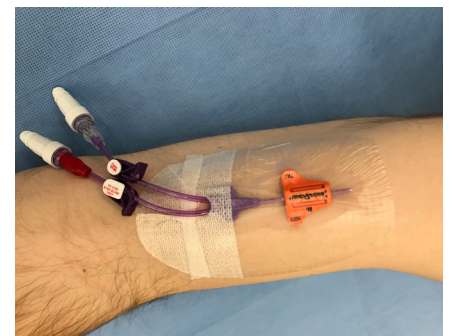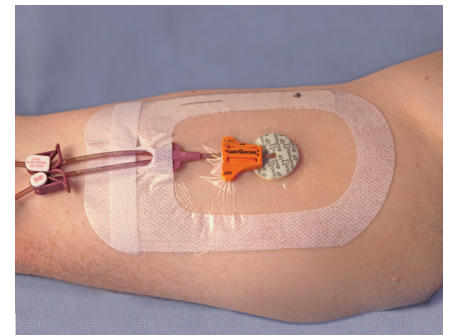

### Distributed By Aquilant

Aquilant House, Unit B1-B2 Bond Close,  
Kingsland Business Park, Basingstoke,  
Hampshire, RG24 8PZ, United Kingdom  
T: +44 (0) 1256 306 506  
E: [contactus@aquilantservices.com](mailto:contactus@aquilantservices.com)

### Interrad Medical

181 Cheshire Lane,  
Suite 100, Plymouth,  
MN 55441 USA  
+1.763.225.6699  
[www.securacath.com](http://www.securacath.com)



### SecurAcath® app!

